

# Oticon ON guide


Create an Oticon ON account and set-up the Oticon ON app to use IFTTT

## What you will need:

Open hearing aid · iPhone 5 (or later) · Oticon ON app

1

Open the Oticon ON app on your iPhone.

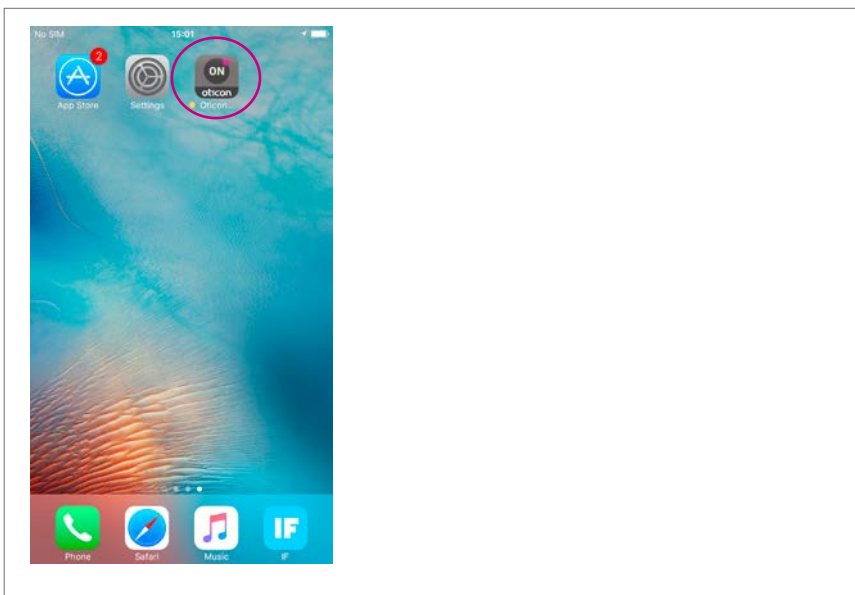

**Note:** If you are launching the Oticon ON app for the first time, you will be asked to share your location. Press 'Allow' if you want to enable 'Find my hearing aids'.


Allow "Oticon ON" to access your location even when you are not using the app?

The app uses location services to help you find your hearing aids.

Don't Allow


Allow

Follow the connection guide and read through the ON app instructions.

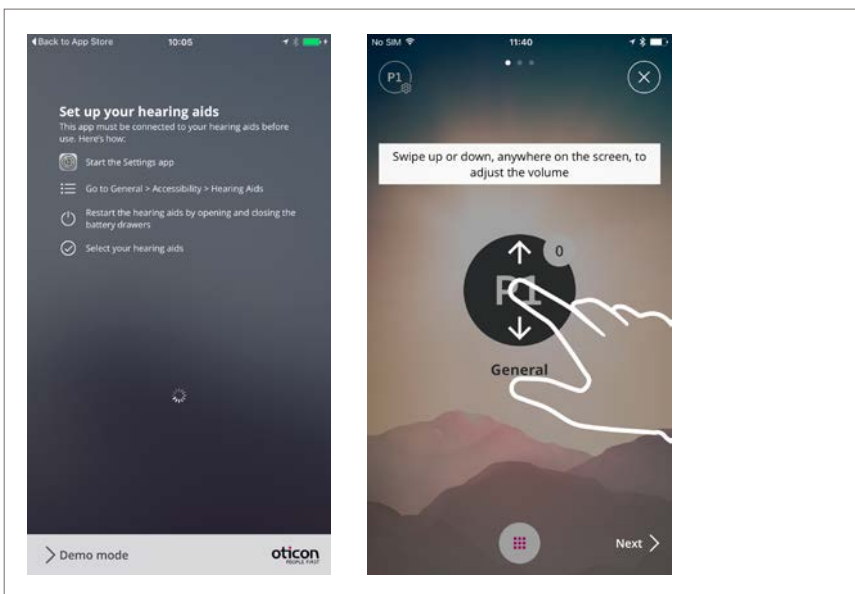


2

Press on the menu button and then click on the 'IF' button.

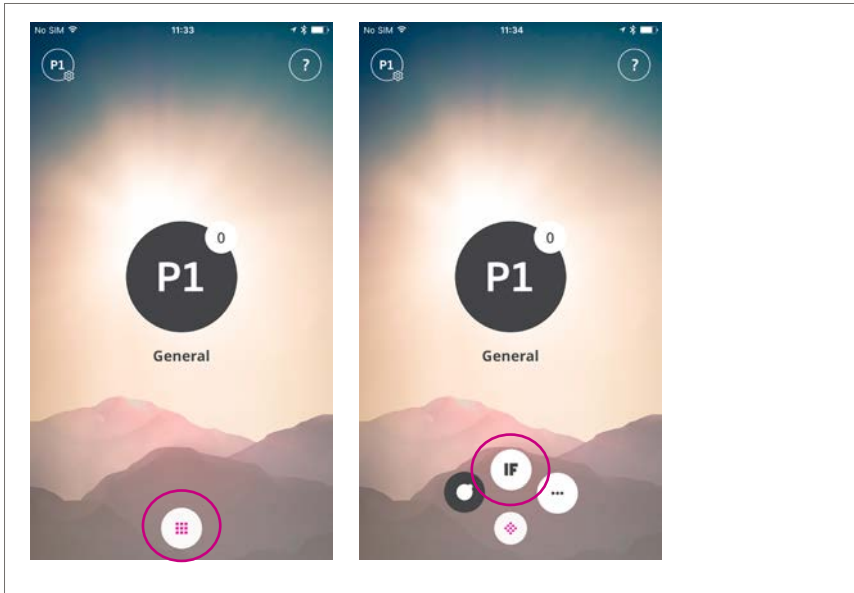

3

Press 'Get started'.

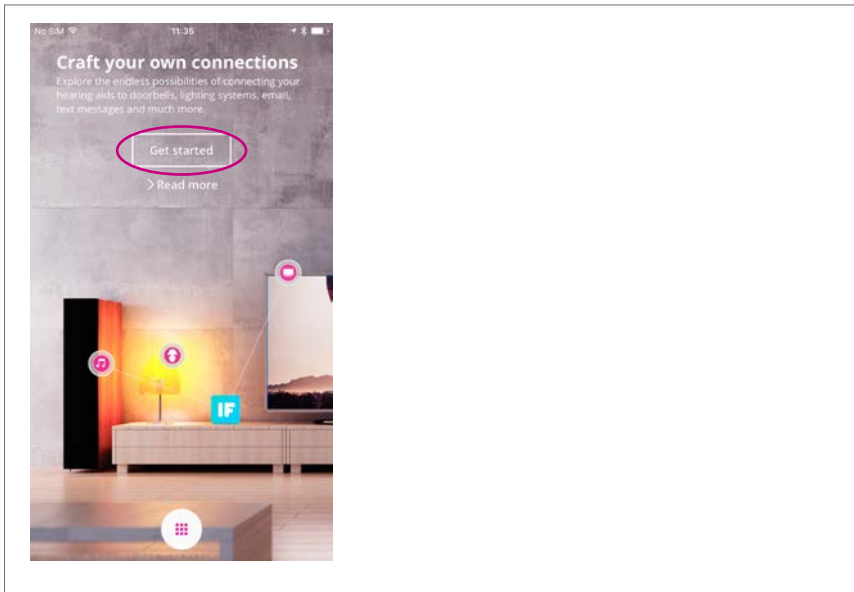

4

If you already have an Oticon On account use it to sign in.  
If you don't have an Oticon On account, you can create one using your Facebook or Google account (see step 4a) or with your email address (see step 4b).

4a

### Sign-up using Facebook or Google

Click on either 'Facebook' or 'Google' and log-in using your account information. Then click on 'accept'.

Cancel

oticon  
PEOPLE FIRST

Sign in with your social account

Facebook Google

OR

Sign in with your existing account

Email Address

Password

Sign in

Don't have an account? Sign up now

Forgotten password

4b

### Sign-up using email

Click on 'Sign up now'.  
Enter your email address and then click on 'Send verification code'.

Cancel

oticon  
PEOPLE FIRST

Sign in with your social account

Facebook Google

OR

Sign in with your existing account

Email Address

Password

Sign in

Don't have an account? Sign up now

Forgotten password

Email Address

Send verification code

New Password

Confirm New Password

Display Name

Create Cancel

Go to your email inbox and open the verification email that has been sent to you.  
Copy the verification code from the email, paste it in to the 'Verification code' field, and click on 'Verify code'.

Verify your email address

Cancel

Thanks for verifying your - your email - account!

Your code is: 147450

Yours sincerely,  
Oticon

Email Address

Verification code

Verify code Send new code

Enter and confirm a new password for your account. Choose a 'Display name' and click on 'Create'.

Cancel

Confirm New Password

Display Name

Create Cancel

5

Confirm that IFTTT service and notifications are enabled. If not enabled, click on the 'IFTTT' and 'Notifications' sliders so they are green.

